## Instructions for hair sampling

### Preparation

#### Required materials:

- ✓ Aluminium foil
- ✓ Scissor
- ✓ Paper clip
- ✓ Hairgrip
- ✓ Comb
- ✓ Packthread loop (see instructions)

### Step 1

- Divide the hair at the back of the head using the hairgrip
- Separate **2-3 hair strands** (**overall diameter similar to half a pencil**) approx. 2 cm below the cranial bone and comb the strands

### Step 2

- Thread the combed strands **through the loop** and **tighten** the loop

### Step 3

- Cut the hair **as close as possible to the scalp**

### Step 4

- Put the strands into **aluminium foil**
- Mark the scalp-near end of the strands with a permanent marker on the aluminium foil

### Step 5

- Fold the aluminium foil
- If the sample is longer than the aluminium foil, hair strands can be cut from the distal end

### Step 6


- Fix the folded aluminium foil at the scalp-near end with a paper clip
- **Do not fold** the sample
- Put the aluminium foil with the hair protocol (including participant ID) in a transparent folder
- Store the sample at a dry and dark place

Examples of *perfect* hair samples (with lots of material to work with...)

This little we really need for a single analysis (5 - 7.5 mg); it is sufficient for either method, immunoassay or LCMS/MS

# What is the Shipping Address???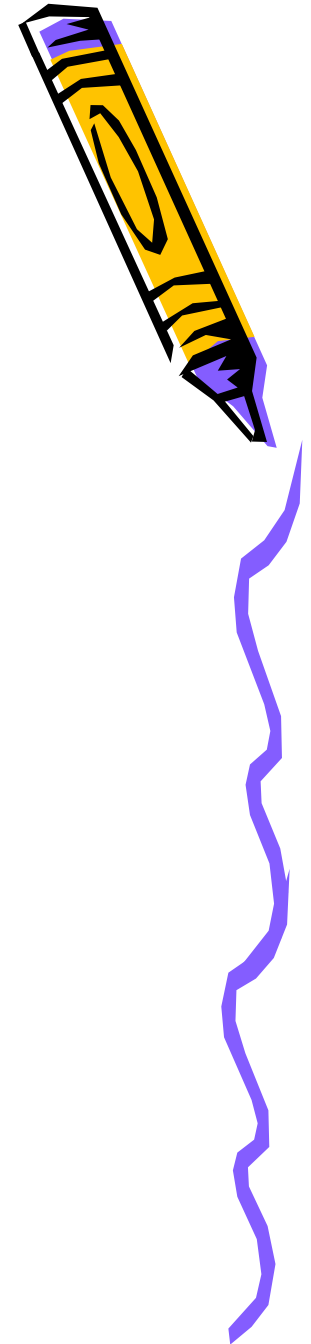

It is:

Prof. Dr. C. Kirschbaum  
Biopsychology - TU Dresden  
Zellescher Weg 19, Raum 212  
D-01062 Dresden, Germany


If you need to provide a contact phone  
number, use: +49-1522-724-735  
or: +49-178-131-6822

# How do I ship my samples to you??


- Use a courier service that will deliver the samples within 3-4 working days (TNT, FedEx, UPS etc.)
- When doing the paperworks, **DO NOT STATE ANY COMMERCIAL VALUE** (write 0.1 \$ in the form; otherwise we will have to pay taxes)
- Name the following **customs tarif number:**

3001 2010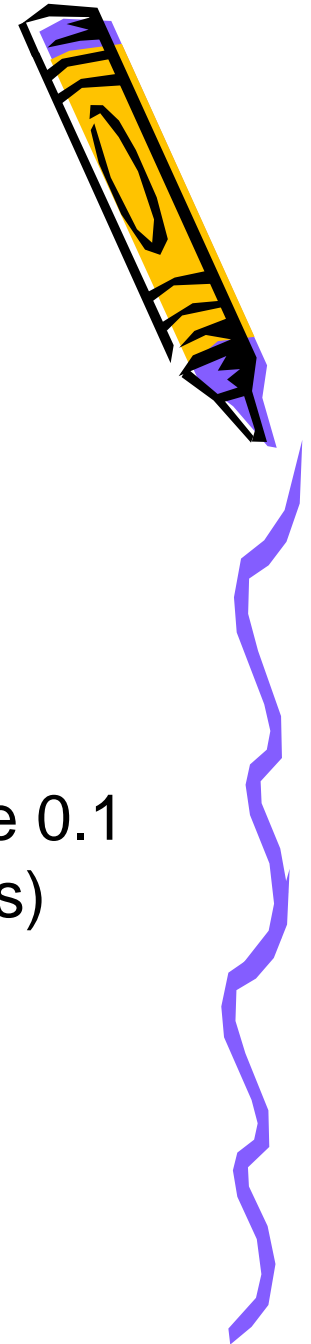

# Avoiding Delays at Customs

- Print out the next page and put the label on top of your shipping box so that the customs people can easily detect it. This usually avoids hold-up of your shipment at customs.

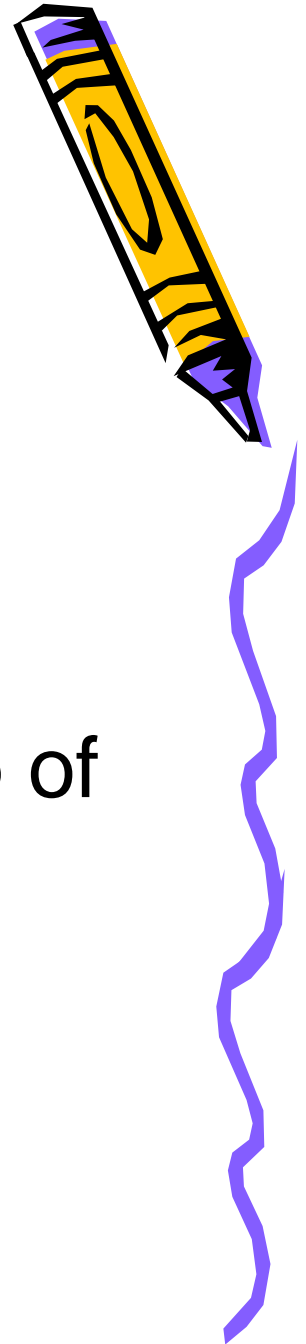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